

# meo<sup>®</sup> plant



## Our new packaging concept

Many advantages  
at the same price!

- ✓ **Double sterile barrier**  
Sterility for 5 years
- ✓ **Improved handling**  
For caregivers and assistants
- ✓ **Direct product removal**  
with screwdriver
- ✓ **Colour-coded labelling**  
A handle for the correct diameter
- ✓ **Sealed outer packaging**  
with practical tear tab

**Meoplant Medical**

GmbH Deutschland

Malchiner Str. 99

12359 Berlin

Phone +49 (0)30 - 755 277 06

Fax +49 (0)30 - 755 277 05

info@meoplant.de

www.meoplant.de

## The procedure – step by step



1 Open secondary packaging via tear tab and remove sterile blister




2 Remove Tyvek foil from the outer blister and take out inner blister



3 Remove Tyvek foil from the inner blister



4 Insert screwdriver directly into the implant and remove

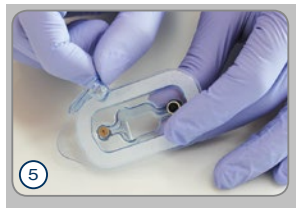

5 Take off protective cover for cover screw



6 Remove cover screw via screwdriver